

Settings

10(A). Manage Your Profile

The Profile section allows you to view your account information and manage your account.

1. Open **Settings** located at the top right corner of your dashboard.
2. Select **Profile** from the navigation panel on the left side.
3. Review your account information, including:
 - Name
 - Email ID
4. Click **Reset Password** if you want to change your password.
5. Click **Logout** to sign out of the application.
6. If required, select **Delete Account** to permanently remove your account.

10(B). View Analysis Limits

The Limits page displays the configured analysis limits used by the application.

1. Open **Settings**.
2. Select **Limits**.
3. Review the configured:
 - RBC Limit - Maximum number of red blood cells considered during analysis.
 - FOV Limit - Maximum number of Fields of View processed for a case.

--



10(C). Configure AI Settings

The AI Settings page allows you to customize how the AI model performs image analysis.

1. Open **Settings**.
2. Select **AI Setting**.
3. Adjust the **Confidence Threshold** using the slider.
4. Select the preferred **Compute Unit** (for example, CPU).
5. The selected settings are applied for future image analysis.

